Instructions for Use

---

LIVINGSKIN<sup>™</sup>

EN		Instructions for Use	3
DE		Gebrauchsanweisung	6
FR		Notice d'utilisation	10
ES		Instrucciones para el uso	14
IT		Istruzioni per l'uso	18
NO		Bruksanvisning	22
DA		Brugsanvisning	25
SV		Bruksanvisning	28
EL		Οδηγίες Χρήσης	31
FI		Käyttöohjeet	35
NL		Gebruiksaanwijzing	38
PT		Instruções de Utilização	42
PL		Instrukcja użytkowania	46
CZ		Návod k použití	50
TR		Kullanım Talimatları	53
RU		Инструкция по использованию	56
JA		取扱説明書	60
ZH		中文说明书	63
KO		사용 설명서	66

# ENGLISH

---

## PRODUCT DESCRIPTION

---

### **LIVINGSKIN™**

LivingSkin™ is a passive functional prosthetic device or i-Limb Skin Match cover for a myoelectric prosthesis.

---

## INDICATIONS FOR USE

---

Individuals with limb absence.

---

## CONTRAINDICATIONS

---

None known.

---

## INTENDED USE

---

Individuals with limb absence and the potential to use a prosthetic device. LivingSkin™ devices are intended for any user with limb absence at the level of the prescribed device.

---

## GENERAL SAFETY INSTRUCTIONS

---

### **Warnings:**

#### **LIVINGSKIN™:**

- Do not use for extreme activities that may cause injury to a natural hand, e.g. rock climbing
- Do not expose to high temperatures
- Do not expose to flames
- Only use water-based lubricants with the device.
- Do not use oil-based lubricants with the device. Using oil-based lubricants will destroy the device and void the warranty.
- Do not use skin care products containing oils with the device.
- Do not use soaps containing oils and/ or additives such as perfumes, glycerin, aloe vera or vitamin E with the device. Exposure to oils and oily substances will make the prosthesis look shiny. Do not use oil control cleansers to manage shine - these are abrasive and will add to the shiny look of the device.
- Do not bath or shower with the device.

### **Precautions:**

- Users must comply with local regulations on the operation of automobiles, aircraft, sailing vessels of any kind and any other motorized vehicle or device. It is entirely the user's responsibility to seek confirmation that they are physically and legally able to drive using the prosthesis and to the fullest extent permitted by law.
  - Never wear the device overnight. Continual use may cause skin irritation.
  - Never wear the device over unhealed cuts or sores.
- 

## CARE AND CLEANING

---

### **Cleaning:**

- Wearers of a LivingSkin passive functional prosthesis should wash their prosthesis after each use. Use warm water and soap, clean inside and out.
- Wearers of a myoelectric prosthetic using a i-Limb Skin Match cover must take care to not allow water inside.
- A small soft nail brush may be used with extreme care.
- Clean your LivingSkin regularly with 70 - 90% isopropyl alcohol to help with disinfection.
- If your skin requires oil or lotion to be applied, wait at least 30 minutes for it to be completely absorbed before donning your prosthesis.

**NOTE:** body perspiration contains oils which can build up and damage the prosthesis if not removed by proper cleaning. Failure to keep your prosthesis clean as instructed above will void the warranty. If you wish to wear gloves, a silk or smooth fabric-lined glove will be much easier to put on and take off.

**Some substances may stain a Livingskin™ device. Avoid the following:**

- blue Woolite®
- dyed or colored shoes
- permanent markers
- some pens and inks (newspaper ink will not stain)
- clothes dye
- unwashed blue jeans and fabrics
- bleach
- chronic exposure to cigar and cigarette smoke
- carbon paper
- “carbonless” carbon paper

Only wear rings made from pure and high-grade metals, e.g. silver and gold, with or on the device.

Make sure leathers and fabrics for covering wrist seams will not bleed fabric dye colors.

If your device is stained, quickly wash it with plain soap and water, then rub with isopropyl alcohol.

If the stain cannot be removed or the device is damaged in any way, contact your prosthetist for advice.

In most cases damage can be repaired.

**NOTE:** do not try to fix the prosthesis yourself. Gluing or trying to repair your prosthesis will void the warranty.

---

## FITTING INSTRUCTIONS

---

If you have received an i-Limb Skin Match cover for a myoelectric prosthesis, please refer to the i-Limb Hand “Instructions for Use” for correct donning and doffing.

For donning a Livingskin passive functional prosthesis, please follow the steps below.

### Donning

Apply a thin layer of water-based lubricant (e.g. Surgilube or any other OTC water-based lubricant) on the appropriate area.

Gently push or pull the device into place, making sure all air has escaped.

**NOTE:** pulling the device from one point can tear the silicone.

- Using too much lubricant can cause the prosthesis to slide or rotate. You will be able to determine the correct amount of lubricant over time and experience of use.
- For finger devices, pinch your skin at the edge of device to create a channel for air to enter or escape.
- For hand devices, secure the device between your knees to make donning easier. Do not tug or pull the wrist section.
- Hard socket prostheses do not usually require lubrication.
- Contact your prosthetist if you experience any discomfort or skin irritation.

### Doffing

Pinch the open end, allowing air to enter and break the suction. Slowly ease the device off.

**NOTE:** tugging or pulling from one point will cause stretching of your device. For a hand device, never remove by pulling the fingers.

If suction is not broken, pinch the open end again until air enters.

Water can be used to help remove the devices. Hold the device under a faucet, pinch the open area and let warm water enter.

To remove from hard socket prostheses, insert a finger or wiggle while gently easing the Livingskin™ device off.

### **Using the Armature**

Livingskin™ devices can be fitted with armatures. Armatures are metal rods inside the prosthesis that allow positioning.

If armatures are included in the finger(s) of your Livingskin™ device, only bend at the knuckle or finger joint closest to the palm. Never bend the armature at the fingernail, as this can make the armature to puncture the surface. If an armature punctures the exterior of a Livingskin™ device, contact your prosthetist to schedule a repair.

**NOTE:** do not constantly use armatures for positioning. Frequently bending the wires will cause them to break. Broken wires cannot be repaired.

### **Battery or Switch Access**

If being used as an arm cover for a myoelectric prosthesis, spray the outside of the Livingskin™ device with isopropyl alcohol before rolling down the cover to charge the battery. Doing so will make it easier to pull the cover against itself, preventing damage that may be caused by friction.

### **Nails**

Acrylic finger or toe nails may be polished as usual. Only use non-acetone nail polish remover. Do not polish silicone nails.

### **Prosthetic Adhesive**

- If required, prosthetic adhesive can be used to aid suspension.
- Apply the adhesive both to the proximal edge of the device and the skin. Let dry. Once contact is made between the adhesive on the skin and the prosthesis a strong bond will be created.
- To doff the prosthesis, wet a cotton bud with the remover. Roll the cotton bud around the edge of the device and work it off the skin.
- If remover is not available, wet a cotton bud with soap. See “*Care and Cleaning*” for information on suitable soaps.
- Please check for an allergic reaction to the adhesive on non-sensitive skin before use. Sensitive formulas are available should skin react allergically.

---

## DISPOSAL

---

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

---

## WARRANTY

---

Livingskin passive/ match devices are covered for 1 year under Touch Care for manufacturing related issues. If a device is returned for repair during the period it is covered under Touch Care and it will be repaired free of charge.

Inappropriate use of the prosthesis resulting in damage is not covered by the warranty.

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**EN – Caution:** Ossur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standards is achieved only when Ossur products are used with other recommended Ossur components. If an unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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